




INTENDED USE

Richmond Dental and Medical's Soft BiteBlock™ is used to help protect the airway device (endotracheal tube or laryngeal mask airway), teeth, and tongue from being bitten when patients emerge from anesthesia. The Soft BiteBlock™ is provided non-sterile and in bulk size large, item no. 260000.

INSTRUCTIONS FOR USE

- Wash your hands and use approved medical gloves
- Assess patient's oral capacity and condition to ensure it is compatible with the Soft BiteBlock™
- Carefully, place the Soft BiteBlock™ between the back molars with the tongue medial
- Make sure the Soft BiteBlock™ protrudes outside the mouth and is visible
- Check to ensure the Soft BiteBlock™ is not pressed against the soft tissue, tonsils, or roof of the mouth to cause injury
- Recheck location of Soft BiteBlock™ frequently during use
- Extract Soft BiteBlock™ when no longer needed
- For single patient use only

Item #	(A) Soft BiteBlock™ Size (Before compression testing)	Bulk Packaging	Wet* (B) Compressed Dia. (mm)	Dry** (B) Compressed Dia. (mm)
260000	Large: 4" (101.6 mm) x 3/4" (19.05 mm)	10/band; 12 bands per box	5	7

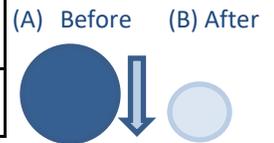

*Wet Compression testing was done on water saturated samples at 600 Newton's/135 pounds' force pressure

**Dry Compression testing at 600 Newton's/135 pounds' force pressure

WARNING AND PRECAUTIONS

This device should only be used by CRNAs, Anesthesiology Technicians, Anesthesiologists, and other persons having adequate training and familiarity with intubation techniques.

When placing the Soft BiteBlock™ into the patient's mouth, care should be exercised to avoid placing fingers between the bite block and patient's teeth.

The Soft BiteBlock™ is not designed to be reprocessed or reused, and therefore cannot verify that reprocessing can clean and/or sterilize or maintain the structural integrity of the device to ensure patient safety.

PRODUCT DISPOSAL

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state, and federal laws and regulations.

